


TECNIS Eyhance™ IOL

Models DIB00,
DIU150-DIU600

with TECNIS SIMPLICITY™ Delivery System



Anterior side

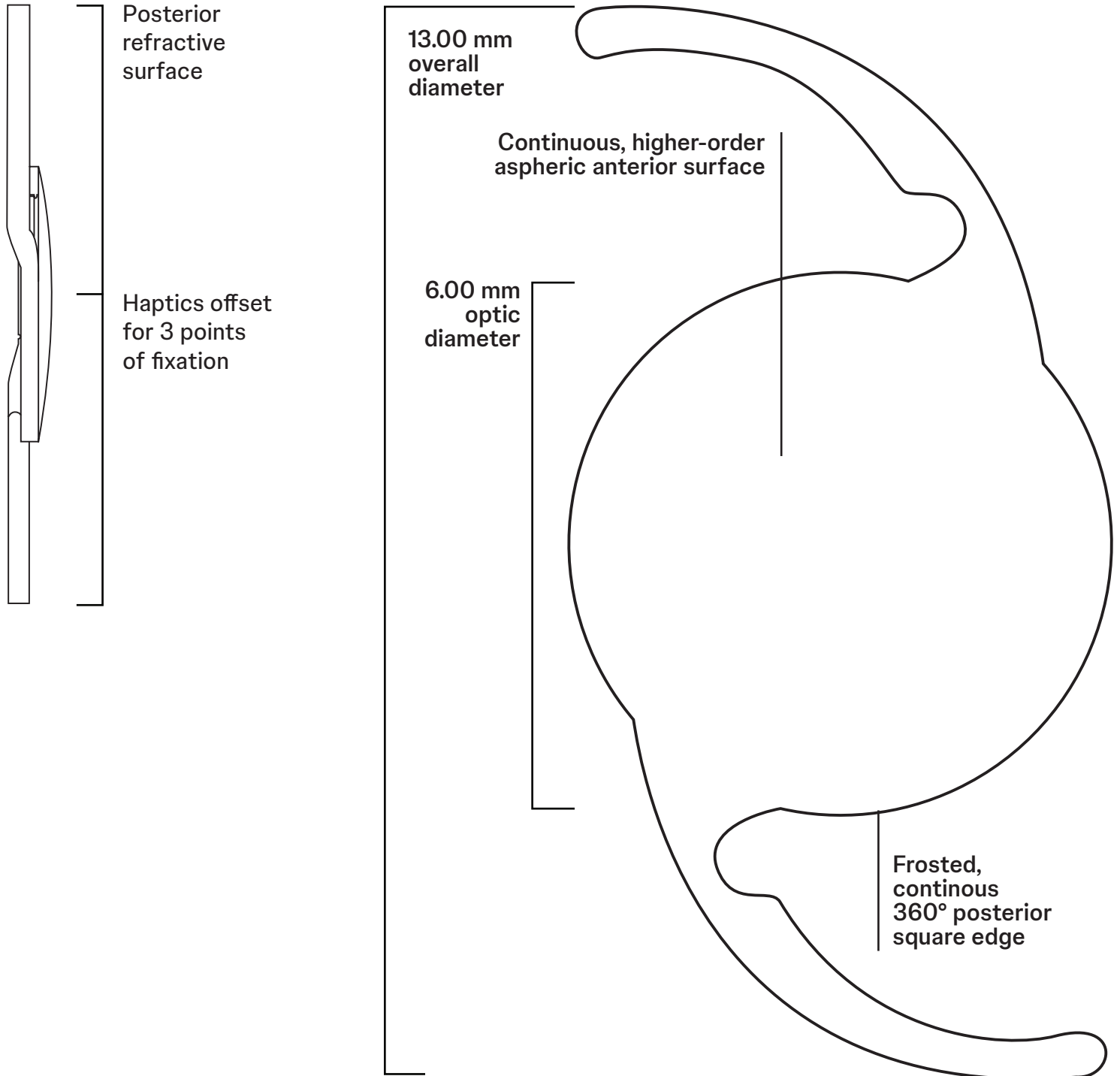

TECNIS Eyhance™ IOL

with TECNIS SIMPLICITY™ Delivery System

Models DIB00,
DIU150-DIU600

Model	TECNIS Eyhance™ IOL	TECNIS Eyhance™ Toric II IOL						
	DIB00	DIU150	DIU225	DIU300	DIU375	DIU450	DIU525	DIU600
Cylinder powers: IOL plane		1.50 D	2.25 D	3.00 D	3.75 D	4.50 D	5.25 D	6.00 D
Cylinder powers: Corneal plane		1.03 D	1.54 D	2.06 D	2.57 D	3.08 D	3.60 D	4.11 D
Powers	+5.0 to +34.0 D in 0.5 D increments							
Diameter	6.00 mm							
Center thickness	0.7 mm (20.0 D)							
Shape	Biconvex, continuous, higher-order aspheric anterior surface							
Material	UV-light absorbing, hydrophobic acrylic							
Refractive index	1.47 at 35° C							
Edge design	ProTEC frosted, continuous 360° posterior square edge							
Delivery system	Preloaded TECNIS SIMPLICITY™ Delivery System							

Biometry*	Contact ultrasound [§]	Optical	Haptic characteristics ³	
A-Constant	118.8	119.3	Overall diameter	13.00 mm
AC depth	5.4 mm	5.7 mm	Thickness	0.46 mm
Lens factor	1.78	2.04**	Style	C
Surgeon factor ¹⁻²	1.68 mm	1.96 mm	Material	UV-absorbing hydrophobic acrylic with violet light filter
			Design	TRI-FIX, Haptics offset from optic; 1-piece lens

*Value theoretically derived for a typical 22.0 D lens.

**This lens factor is based on our manufacturer A-constant. For Barrett optimized lens factor please refer to the Barrett Universal website.

§IOL constants have been theoretically derived for contact ultrasound.

||IOL constants have been derived from clinical evaluation results of the 1-Piece IOL Platform.

Johnson & Johnson Surgical Vision, Inc. recommends that surgeons personalize their A-constant based on their surgical techniques and equipment, experience with the lens model and postoperative results

References:

- REF2022CT4I07 Z311524E_A TECNIS Eyhance™ IOL with TECNIS SIMPLICITY™ Delivery System US DFU.
- REF2021CT4007 Z311525E_A TECNIS Eyhance™ Toric II IOL with TECNIS SIMPLICITY™ Delivery System DFU.
- TECNIS Eyhance™ Toric II IOL with TECNIS SIMPLICITY™ Delivery System, Model DIU - DFU US Z311770E Rev. B. 2024REF4596

INDICATIONS and IMPORTANT SAFETY INFORMATION for TECNIS Eyhance™ and TECNIS Eyhance™ Toric II IOLs with TECNIS SIMPLICITY™ Delivery System

Rx Only

INDICATIONS FOR USE: The TECNIS SIMPLICITY™ Delivery System is used to fold and assist in inserting the TECNIS Eyhance™ IOL for the visual correction of aphakia in adult patients in whom a cataractous lens has been removed by extracapsular cataract extraction. The lens is intended to be placed in the capsular bag. The TECNIS SIMPLICITY™ Delivery System is used to fold and assist in inserting the TECNIS Eyhance™ Toric II IOLs for the visual correction of aphakia and pre-existing corneal astigmatism of one diopter or greater in adult patients with or without presbyopia in whom a cataractous lens has been removed by phacoemulsification and who desire reduction in residual refractive cylinder. The lens is intended to be placed in the capsular bag. **WARNINGS:** Physicians considering lens implantation should weigh the potential risk/benefit ratio for any conditions described in the Directions for Use that could increase complications or impact patient outcomes. The lens should be placed entirely in the capsular bag. Do not place the lens in the ciliary sulcus. Rotation of the TECNIS Eyhance™ Toric II IOL from its intended axis can reduce its astigmatic correction. Misalignment greater than 30° may increase postoperative refractive cylinder. If necessary, lens repositioning should occur as early as possible, prior to lens encapsulation. Do not attempt to disassemble, modify or alter the delivery system or any of its components, as this can significantly affect the function and/or structural integrity of the design. Do not implant the lens if the rod tip does not advance the lens or if it is jammed in the delivery system. The lens and delivery system should be discarded if the lens has been folded within the cartridge for more than 10 minutes. **PRECAUTIONS:** The safety and effectiveness of the TECNIS Eyhance™ IOL has not been substantiated in clinical trials and the effects of the optical design on quality of vision, contrast sensitivity, and subjective visual disturbances (glare, halo, etc.) have not been evaluated clinically. This is a single use device, do not resterilize the lens or the delivery system. Do not store the device in direct sunlight or at a temperature under 5°C (41°F) or over 35°C (95°F). Do not autoclave the delivery system. Do not advance the lens unless ready for lens implantation. The contents are sterile unless the package is opened or damaged. The recommended temperature for implanting the lens is at least 17°C (63°F). The use of balanced salt solution or viscoelastics is required when using the delivery system. Do not use if the delivery system has been dropped or if any part was inadvertently struck while outside the shipping box. **ADVERSE EVENTS:** The most frequently reported cumulative adverse event that occurred during the SENSAR™ 1-Piece IOL clinical trial was cystoid macular edema which occurred at a rate of 3.3%. **ATTENTION:** Reference the Directions for Use for a complete listing of Indications and Important Safety Information.

For precise results, utilize the TECNIS™ Toric Calculator to determine the appropriate Toric model and power. Based on preoperative keratometry, biometry, and surgeon preferences, the calculator provides three IOL options, with residual astigmatism, to assist surgeons in accurate lens model selection and axis placement. www.TecnisToricCalc.com

Johnson & Johnson

For healthcare professionals only. Please reference the Instructions for Use for a complete list of Indications and Important Safety Information and contact our specialists in case of any question.

© Johnson & Johnson and its affiliates 2025

2025PP18421